
Biofourmis Care

Medications (Web Dashboard)




Step 1

To access a study participant's "Medications" page, search for the study participant or scroll down the patient list. Click their name to open their "Patient details" page.

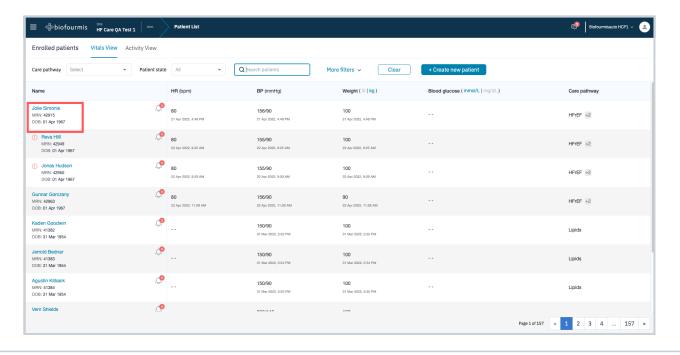

Step 2

From the "Patient details" page, click on the "Medications" tab. The medications page appears with the active medications, discontinued medications, and the ability to add/edit medications.

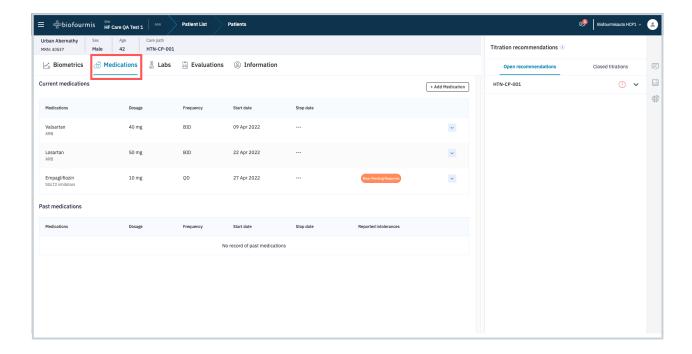


Biofourmis Care

Medications (Web Dashboard)




Add a New Medication

To add a new medication, click "Add a new medication" on the "Medications" page. Fill in the required information and then click "Add new medication".

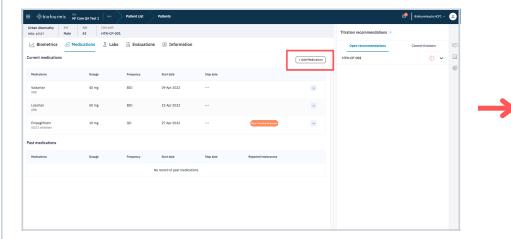


Note: Any updates to medications (new medications, adjustments, and discontinuations) must be acknowledged by the study participant through the patient application.

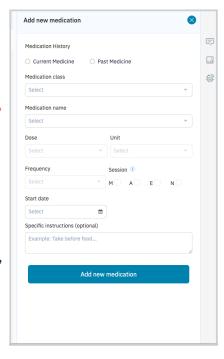

Medication Compliance

The "Medications" page will display the study participant's compliance with their medications. Compliance is self-reported by the study participant through the patient application. To view compliance, click the arrow next to a specific medication.

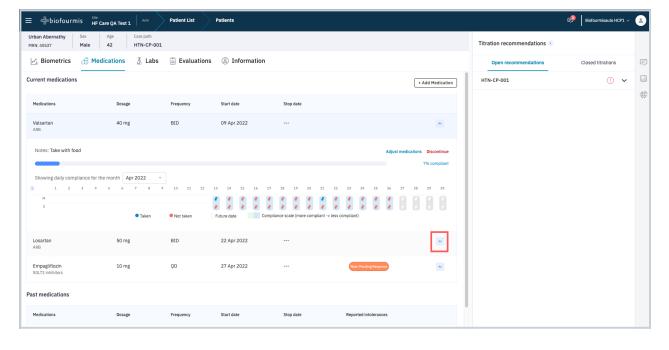


Biofourmis Care

Medications (Web Dashboard)



Titration Recommendations

Titration recommendations will display on the "Medications" page in "Patient details". New changes to the study participant's medication will display within the titration recommendation window on the far right of the screen. You are able to accept or decline these recommendations for a study participant.

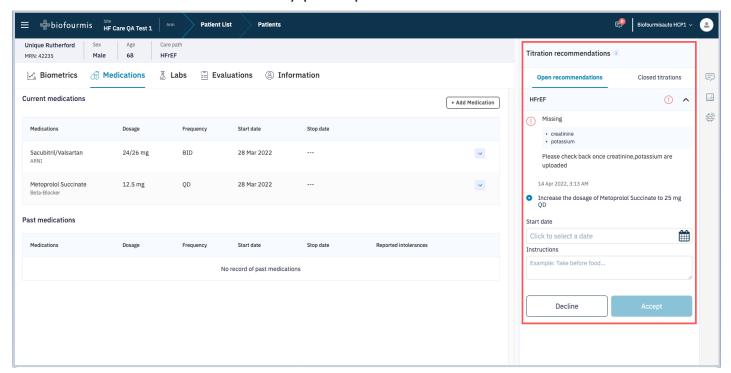

There are two ways to respond to a titration recommendation:

- 1) Accept the titration recommendation: To accept the titration recommendation, indicate the start date for the updated medications along with any additional notes for the study participant.
- **2) Decline the recommendation:** Using your clinical judgment, you can decline the recommendation if the titration recommendation does not need to be added to the study participant's care plan. If you decline a titration recommendation, the Biofourmis Care dashboard will provide alternative recommendations.